Tubing Wrap

STANDARD AND SLIMLINE

User Instructions

English ● Deutsch ● Français ● Italiano ● Español ● Português ● Nederlands ● Svenska ● Dansk ● Norsk ● Suomi
● 简体中文 ● 繁體中文

English

The tubing wrap is intended to reduce condensation and temperature loss in ResMed air tubing when connected to a device.

The tubing wrap is available in two variants, Standard (33963) and SlimLine (36811), referred to collectively as 'tubing wrap' throughout this manual.

Code	Colour	Length	Diameter
33963	Blue	6'6" (2 m)	0.75" (19 mm)
36811	Grey	6' (1.8 m)	0.6" (15 mm)

Fitting

1. Fully unzip the tubing wrap and unfasten the Velcro™ strap.
2. Lay the tubing wrap down with the inside facing up.
3. Pass one end of the air tubing through the closed zipper end of the tubing wrap until the rubber cuff is through.
4. Hold the rubber cuff in position, then zip up the tubing wrap.
5. Fold the extended end of the zipper back against the tubing wrap and secure using the Velcro strap. Only the rubber cuffs should now be visible.

Note: If you have a Standard tubing wrap and you want to use it with SlimLine air tubing, you must complete this additional step between steps 3 and 4:

- At the open end of the tubing wrap, fold the extra length over by approximately 6" (15 cm) and secure using the Velcro.

WARNING

Do not use the tubing wrap if the room temperature is above 86°F (30°C). At higher temperatures the air temperature at the mask may be too hot.

Note: For any serious incidents that occur in relation to this device, these should be reported to ResMed and the competent authority in your country.

Cleaning instructions

The tubing wrap should be hand washed or dry cleaned only. Wash the tubing wrap separately when new, as the colour may run. Do not bleach.

To wash the tubing wrap:

1. Wash in cold to warm water using a mild detergent.
2. Rinse well and drip dry.

Note: Hand washing your tubing wrap does not replace the need to clean your air tubing.

Disposal

The tubing wrap and packaging does not contain any hazardous substances and may be disposed of with your normal household refuse.

Symbols

The following symbols may appear on your tubing wrap or packaging.

-  Warning;  Consult instructions for use;
-  Manufacturer;  European Authorised Representative;  Catalogue Number;  Batch Code;  Importer;  Medical device. See symbols glossary at www.resmed.com/symbols

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Velcro is a trademark of Velcro Industries B.V. © 2020 ResMed. 3681089/2 2020-11
Manufacturer: ResMed Pty Ltd 1 Elizabeth Macarthur Drive Bella Vista NSW 2153 Australia.